




Only with Neoss!

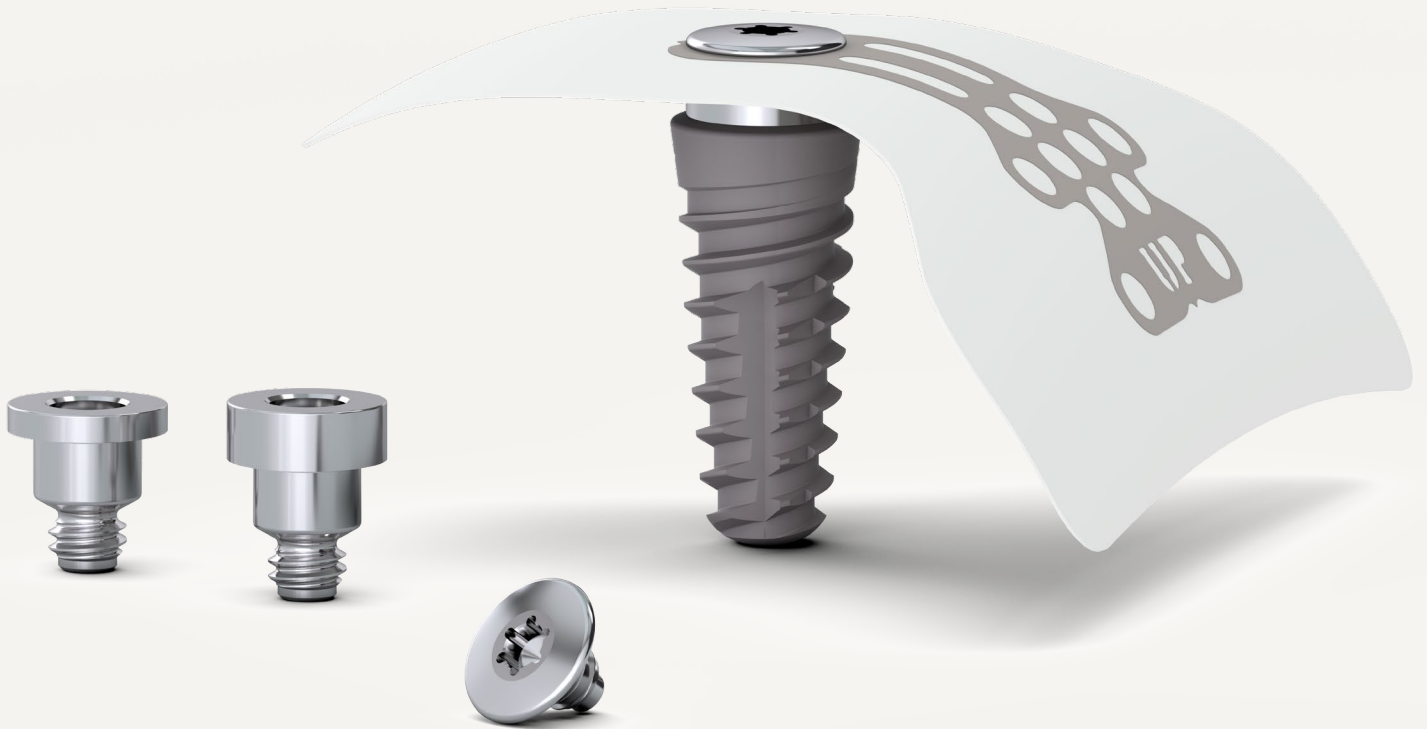

Secure your *vertical bone growth*

NeoGen® Cape PTFE Membranes

Intelligent Simplicity

A simple solution for *predictable vertical bone growth*

The new single tooth PTFE implant-secured membrane has its uniqueness in the main anatomical geometry as well as in the way of securing it on to the Neoss implants.

The membranes contours are tailored to minimize the need for manual outer shape adjustment.

- ✓ Fewer treatment steps thanks to direct fixation to the implant
- ✓ Helps you with buccal bone deficiencies in the esthetic zone
- ✓ Flexibility in height with two spacer sizes for controlled bone-regeneration height

NeoGen® Cape Screw

Clamps the membrane on top of the spacer, one type fits all spacer configurations

NeoGen® Cape PTFE Membrane

Titanium reinforced membrane designed with pre-punched hole for implant fixation

NeoGen® Cape Spacer

Fits into the dental implant connection i.e., unique implant connection



Christian Schober, DDS
Specialist in maxillofacial surgery
Vienna, Austria

Clinical case

NeoGen® Cape PTFE membrane fixed on a simultaneously placed implant.

(See full case series in the Webinar.)



Scan QR code
to see full case series.

Clinical
case

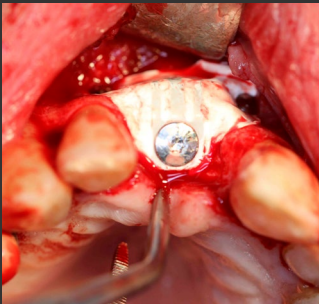


Figure 1. NeoGen Cape PTFE membrane fixed on a Neoss ProActive® implant to treat a buccal bone defect.

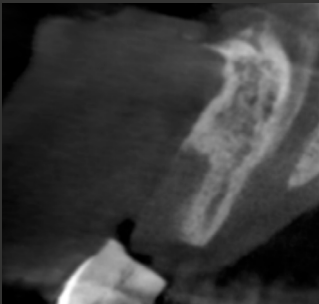


Figure 2. CBCT image of the initial situation.

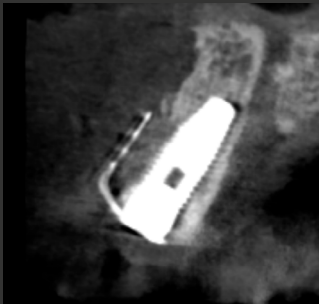


Figure 3. Postoperative situation with implant and NeoGen Cape PTFE membrane in place.


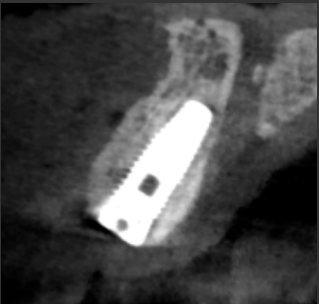


Figure 4. Outcome after five months healing. Note the regenerated buccal bone.

Assortment



Art. No.	Description	A	B	C	D
64040	NeoGen Cape PTFE Membrane – M	28mm	30mm	16mm	9mm
64041	NeoGen Cape PTFE Membrane – L	28mm	30mm	21.6mm	9mm
64080	NeoGen Cape Spacer 0.7 mm SP				
64081	NeoGen Cape Spacer 1.5 mm SP				
64082	NeoGen Cape Spacer 0.7 mm NP				
64083	NeoGen Cape Spacer 1.5 mm NP				

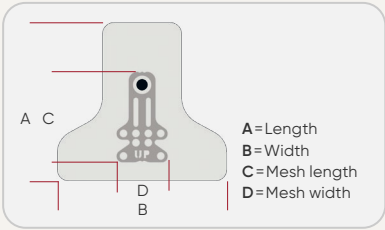

Scan QR code to see full assortment of NeoGen Cape and PTFE Membranes.

UNITED KINGDOM & IRELAND OFFICE
Neoss UK & Ireland Ltd
Windsor House, Cornwall Road, Harrogate, HG1 2PW
T: +44 1423 817 733
E: info@neoss.com

Contact your local representative for product availability details. All products are not available in every market.

15410_0 EN 23-02 © Neoss Group, 2023. Copyrights, design rights and trademarks, Neoss documents, software and designs may not be reprinted, copied or published in whole or part, without the written authorization of Neoss. Neoss, the N logo and NeoScan™ are trademarks of Neoss Group.