Installation and Operation

NeoScan™ 1000

Customer support

For support, please visit our webpage:
neoss.com/neoscan1000

Email support

UK: neoscan.support.uk@neoss.com
US: neoscan.support.us@neoss.com
DE: neoscan.support.de@neoss.com
IT: neoscan.support.it@neoss.com
SE: neoscan.support.se@neoss.com
NO: neoscan.support.no@neoss.com
ES: neoscan.support.es@neoss.com
FR: neoscan.support.fr@neoss.com
DK: neoscan.support.dk@neoss.com
NL: neoscan.support.nl@neoss.com
Other: neoscan.support@neoss.com

NeoScan™ 1000

Notice

The information in this document is subject to change. Neither Neoss AB nor Neoss Ltd or any of its subsidiaries shall be liable for errors contained herein or for incidental damages in conjunction with the furnishing, performance, or use of this material. No part of this publication may be reproduced without the permission of the manufacturer.

All trademarks and registered trademarks are the property of their respective holders.

The NeoScan 1000 is intended for professional use only.

U.S. Federal law restricts this device to sale by or on the order of a dentist.

Manual Name: NeoScan 1000 User Manual: Installation and Operation

Part Number: 15101

Revision: 0

Date: 22-09

Customer support

UK: neoscan.support.uk@neoss.com

US: neoscan.support.us@neoss.com

Other: neoscan.support@neoss.com

Contents

1. Safety information		5
2. NeoScan 1000 Overview	Conventions in this guide	5
	NeoScan 1000 Components	5
	Status indicator overview	6
	NeoScan 1000 holder overview	6
	NeoScan 1000 packing list	6
3. NeoScan 1000 Software Overview		7
	Computer system requirements	7
	NeoPro interface overview	7
	Toolbar overview	7
	Toolbar overview - <i>continued</i>	9
	Option menu overview	10
	Import/Export Scan Data overview	10
	Scan History dialog overview	10
	Preference dialog overview	10
	About dialog overview	12
4. Setting up the scanner		13
	Setting up the scanner	13
	Using the scanner holder	14
	Installing the desktop holder	14
	Installing the wall mount holder	15
5. Getting started		16
	Accessing the NeoPro user interface	16
	Audio prompts	17
	Preparing the teeth	17
	Preparing the scanner	17
	Starting scanning	18
	Scanning approach	18
	Scanning protocol	18
6. Acquiring a 3D model using the common scan workflow		19
	Scanning the upper and lower jaw	19
	Scanning the buccal bite registration	21
	Refining and checking the 3D model	22
	Completing and saving the 3D model	23
7. Acquiring a 3D model using the scan body workflow		23
	Scanning the upper and lower jaw	24
	Scanning the buccal bite registration	25
	Marking the implant areas	26
	Installing and scanning the scan body	27
	Refining and checking the 3D model	28
	Completing and saving the 3D model	29
8. Acquiring a 3D model using the preparation workflow		30
	Import the scanned data before preparing	30
	Scan the preparation areas	31
	Refining and checking the 3D model	33
	Completing and Saving the 3D Model	34
9. Maintenance		35
	Cleaning, disinfecting, sterilizing	35
10. Troubleshooting		36
	NeoScan 1000 troubleshooting instructions	36

1. Safety information

Conventions in this guide

The following special messages emphasize information or indicate potential risks to personnel or equipment.

WARNING: Warns you to avoid injury to yourself or others by following the safety instructions precisely.

Note: Alerts you to a condition that might cause serious damage.

Important: Alerts you to condition that might cause problems.

Tip: Provides extra information and hints.

2. NeoScan 1000 Overview

NeoScan 1000 is designed to acquire 3D models in the followings:

- Upper jaw
- Lower jaw
- Buccal bite registration

NeoScan 1000 Components

NeoScan 1000 consists of the following parts:

1. Reusable tip
2. Start / stop scan button
Press once to start scanning
Press again to stop scanning
3. Mode indicators

Upper jaw scan mode

Lower jaw scan mode

Buccal bite registration mode

4. Mode button & status indicator
Press this button to switch between different modes.
5. USB cable
6. USB connector

Status indicator overview

The relationship between the status of the scanner and the status indicator is as follows:

Scanner Status	Status Indicator
Power off	Off 
Connected	Solid blue 
Startup	Flashing 
Sleep	Breathing light 

NeoScan 1000 holder overview

Place the scanner in the holder when you are not using it.

The scanner goes into sleep mode when put into the holder or left idle for 10 seconds. To use it again, take it out of the holder or pick it up and press any button on the scanner.

The scanner will automatically shut down if it is idle for more than 10 minutes. If you need to use it again, press any button on the scanner, or take it out of the holder.

NeoScan 1000 packing list

Part	Quantity
Scanner handpiece	1
Reusable tip	4
Holder set	1
Screw pack for holder	1
NeoScan 1000 user manual: Installation and operation	1
NeoScan 1000 user manual: Safety, regulatory and technical specifications	1

3. NeoScan 1000 Software Overview

Computer system requirements

For the computer system requirements, see the **NeoScan 1000 User Manual: Safety, Regulatory and Technical Specifications**.

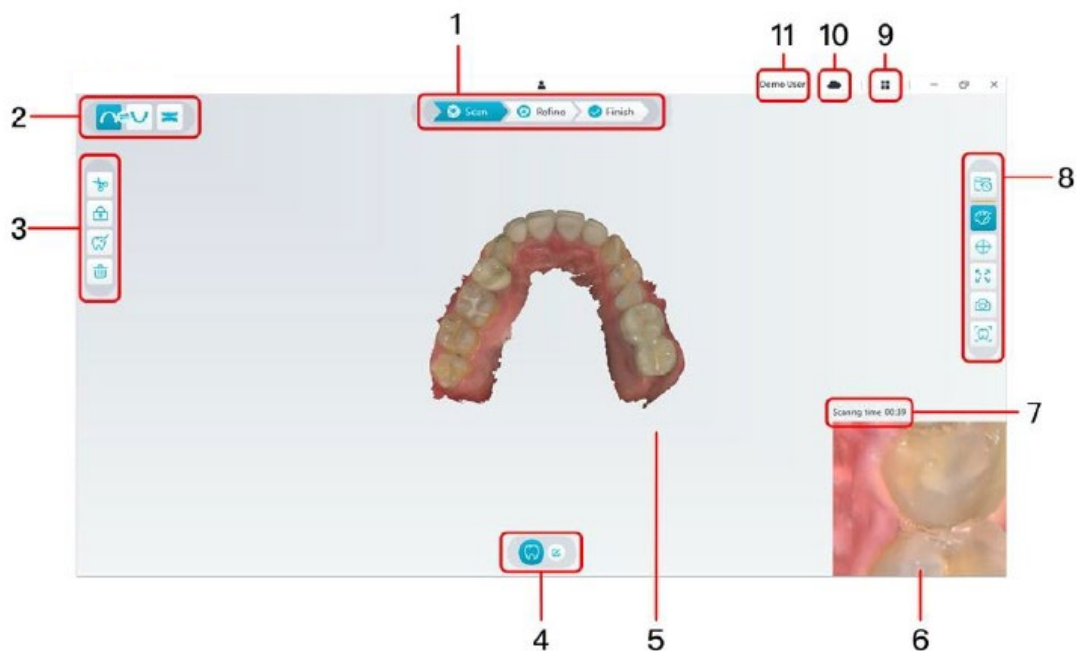
It is **MANDATORY** to check that your computer system configuration is compatible with the computer system requirements for the NeoScan 1000 software.

NeoScan 1000 interface overview

The NeoScan 1000 scanner operates with NeoPro software. The NeoPro interface enables you to acquire 3D models in two ways:

- Partial arch scan: Several teeth in the preparation area on both the upper and lower jaws, and buccal bite registration.
- Full arch scan: Upper jaw, lower jaw, and buccal bite registration

1. **Acquisition Step:** Displays the current step in the acquisition process.
2. **Jaw/Bite Switch:** Enables you to select the upper jaw, lower jaw, or buccal bite registration.

3. **Image Toolbar:** Enables you to select and manage 3D images.
4. **Workflow Toolbar:** Enables you to choose general scan, preparation scan and scan body scan.
5. **3D Model Display Screen:** Displays the 3D model created from the scan.
6. **Video Preview Screen:** Displays live video when scanning, or the scanner status when not scanning.
7. **Scanning Time:** Displays the total scanning duration of the current case.
8. **Display Toolbar:** Enables you to change how the 3D model is displayed.
9. **Option Menu:** Access the system configuration and version information.
10. **NeoConnect Service:** You can view the cloud service status.
11. **Login Menu:** Enables you to log in, switch users and access NeoConnect.

Toolbar overview

Acquisition Step Toolbar

-  **Scan** button: Enables you to scan the upper and lower arch, and the buccal bite registration.
-  **Refine** button: Refines the acquired 3D model, and enables you to use various tools to check the refined results.
-  **Finish** button: complete the case information and save the scan results.

Jaw/Bite Switch Toolbar

-  **Upper Jaw** button: Acquires a 3D model of the upper jaw.
-  **Lower Jaw** button: Acquires a 3D image of the lower jaw.
-  **Buccal Bite Registration** button: Acquires a 3D image of the bite.
-  **Switch Upper/Lower Jaw** button: Changes the acquisition mode from upper to lower or vice versa, if you accidentally scan teeth on the wrong jaw.

Image Toolbar

-  **Cut** button: Draw a curve to delete unnecessary data.
-  Undo last cut
-  Return to the upper level tool (icon on the left)
-  **Lock** button: Enables you to select and lock an area on the model to prevent it from being updated by additional scanning.
-  Unlock the last locked area
-  Return to the upper level tool (icon on the left)
-  **Mark tooth** button: Mark one or more preparation / implant areas.
-  Delete marked teeth
-  Return to the up-level toolbar (icon on the left)

-  **Delete** button: Delete all models from the current case.
-  **Quadrant Snapshot** button: Displays a preview of five 2D images showing different views of the model.
-  **Transparency** button: Set the transparency of the scanned model.

-  **Undercut** check button: View the undercut areas on the surface of the preparation.
-  **Occlusion analysis** button: Analyze the occlusion space.

Toolbar overview - continued

Display Toolbar

True Color button: When selected, displays the 3D model in the actual color of the patient's hard and soft tissues. When deselected, displays the 3D model in monochrome.

Zoom fit button: Scales the 3D model to the best size to fit the display region.

View orientation button: Displays a list of views.

Front view

Back view

Left view

Right view

Top view

Bottom view

Return to the up-level toolbar (icon on the right)

Snapshot button: Takes a snapshot of the 3D model as it appears on your screen.

Intraoral Camera button: Enables you to select 2D intraoral images.

Workflow Toolbar

Common Scan button: Enables you to acquire a 3D model of the upper jaw, lower jaw, and the buccal bite.

Config Scan button: Edit acquisition type.

Add preparation scan button: Add a preparation scan step.

Add scan body button: Add a scan body scan step.

Preparation button: Activate the preparation scan.

Scan body button: Activate the scan body scan.

Option Menu

Option menu button: Enables you to access Import/Export scan data, Scan history, Preference and About dialogs.

Scanner Status Icons

Scanner is not connected

Scanner is connecting

Scanner is overheating

Scanner is in sleep mode

Scanner is in the holder

Scanner tip is not detected

Scanner is in pause state

Option menu overview

Import/Export Scan Data overview

The import/export scan data menus allow users to export the current scan data to a specified folder, and import it later to continue scanning or perform other operations.

If you need to continue scanning after importing the previously saved scan data, you must ensure that the scan data is acquired by the same scanner currently connected, otherwise you will not be able to perform subsequent scans on the imported data.

Scan History dialog overview

The Scan History dialog allows users to import automatically saved scan history data. Based on different user settings, the scan history data is retained for up to 30 days. In the scan history dialog, you can search data by ID or name. To import or manage the selected record, right-click the entry, and then select "Open", "Lock/Unlock" or "Delete".

Open: Import the selected scan history data.

Lock/Unlock: Lock or unlock the selected scan history. The locked scan history records will not be cleared after the storage period expires.

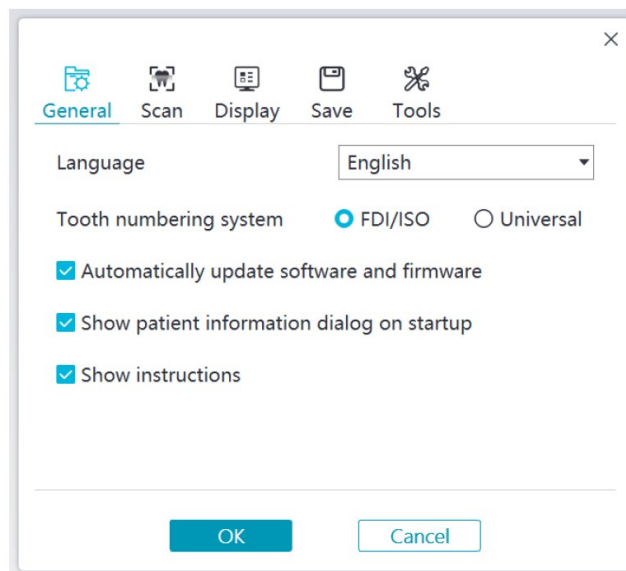
Delete: Delete the selected scan history data.


If you need to continue scanning after importing the previously saved scan data, you must ensure that the scan data is acquired by the same scanner currently connected, otherwise you will not be able to perform subsequent scans on the imported data.

Preference dialog overview


The Preferences dialog enables you to select the software and scanner settings.


- **Language:** Select the user interface language.
- **Tooth numbering system:** Select the FDI/ISO or Universal as the tooth numbering system.
- **Automatically update software and firmware:** When selected, the software will automatically prompt of new versions.
- **Show patient information dialog on startup:** When selected, the patient information dialog will pop up when NeoPro opens.
- **Show instructions:** When selected, the software will show tutorial animation.

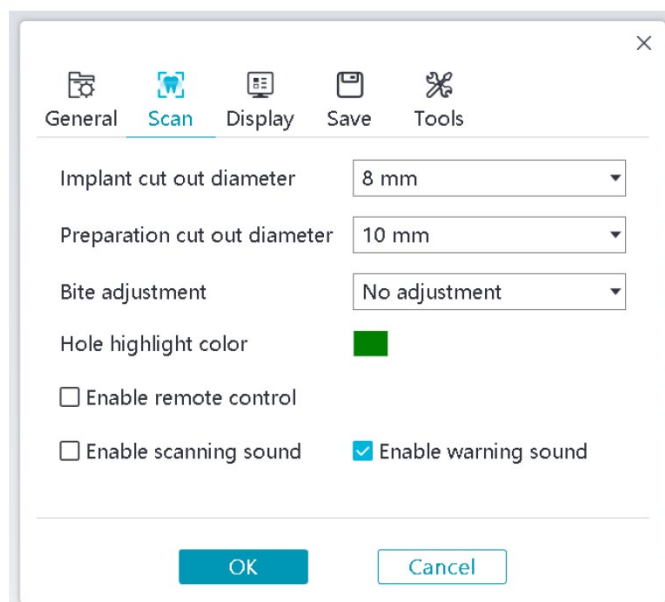
- **Implant cut out diameter:** Set the diameter of the automatic cut out areas in implant mode, it is recommended to choose a value slightly larger than the diameter of the scan body used.
- **Preparation cut out diameter:** Set the diameter of the automatic cut out areas in preparation scan mode, it is recommended to choose a value slightly larger than the preparation diameter.
- **Bite adjustment:** Set whether to adjust the bite result to avoid intersection.
- **Hole highlight color:** Fill the holes in the model with the specified color after refining.
- **Enable remote control:** When selected, the scanned 3D model can be rotated by rotating the scanner after the scanning is stopped.
- **Enable scanning sound:** When selected, a sound will be continuously played when you are successfully scanning (if your computer does not have speakers, this option will not take effect).
- **Enable warning sound:** When selected, a warning sound will be played if the scanning duration exceeds the recommended thresholds, a strong light is detected, or the scanning performance is declined (if your computer does not have speakers, this option will not take effect).

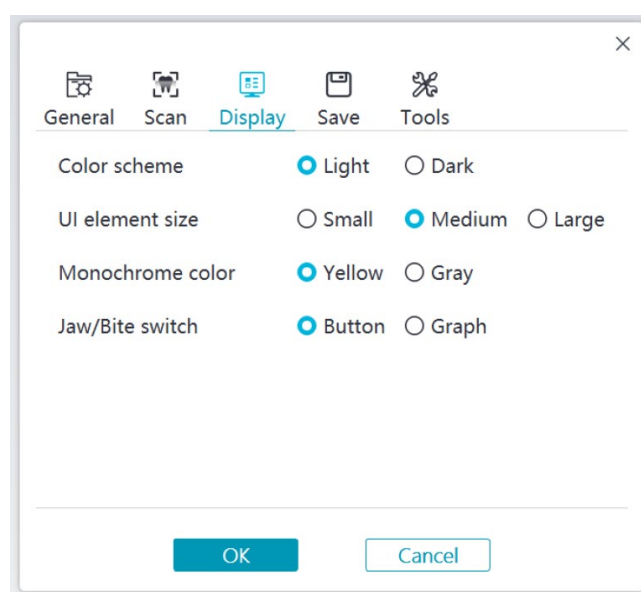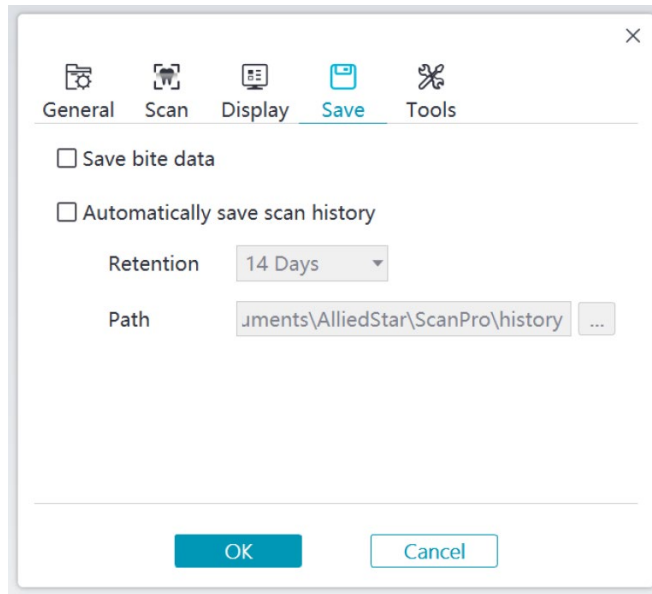- **Color scheme:** Select the color scheme of the user interface.
- **UI element size:** Select the size of interface icons to adapt to different screen resolutions.
- **Monochrome color:** Select the color when displaying the 3D model in monochrome.
- **Jaw/Bite switch:** Select the style of jaw/bite switch.

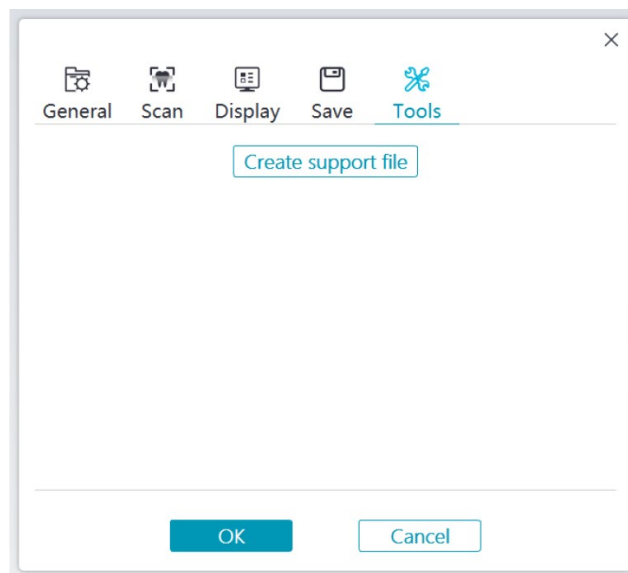- **Save bite data:** When selected, save the bite images to separate files.
- **Automatically save scan history:** When selected, scan history is automatically saved when the software is closed. When this option is enabled, the user can customize the number of days and the path to save the scan history.

- **Create support file:** Export software and firmware logs.

About dialog overview

The About dialog displays software version, firmware version and scanner serial number information.

Example

4. Setting up the scanner

Setting up the scanner

To set up the scanner, follow these steps:

1. Visit www.neoss.com and download the installation file and user manual according to the product model.
2. Double-click the software installation file SetupNeoPro_x.x.x.x.exe (where x.x.x.x represents the internal version number of the NeoPro software)
3. Choose a language from the drop-down list and click Ok to install
4. Follow the instructions on the screen to complete the installation
5. Firmly slide one of the tips onto the end of the scanner.

6. Insert the USB connector of the scanner to any USB 3.0 port on the computer

Make sure the scanner is connected to the USB 3.0  port. If connected to a USB 2.0 port, the scanner may not work properly.

7. The scanner will automatically turn on, the status indicator flashes slowly, and then enters the sleep state, waiting for connection.

When the scanner is connected to NeoPro, the indicator stays in solid blue.

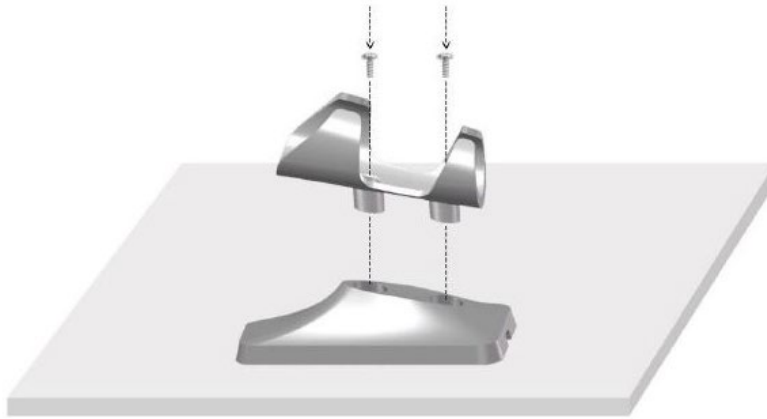
Using the scanner holder

The holder can function either as a desktop holder or a wall mount holder.


Installing the desktop holder

To use the holder as a desktop holder, follow these steps:

- Select a clean surface area
- Place the base of the desktop holder on the desktop, and fix the holder bracket on the base with screws.

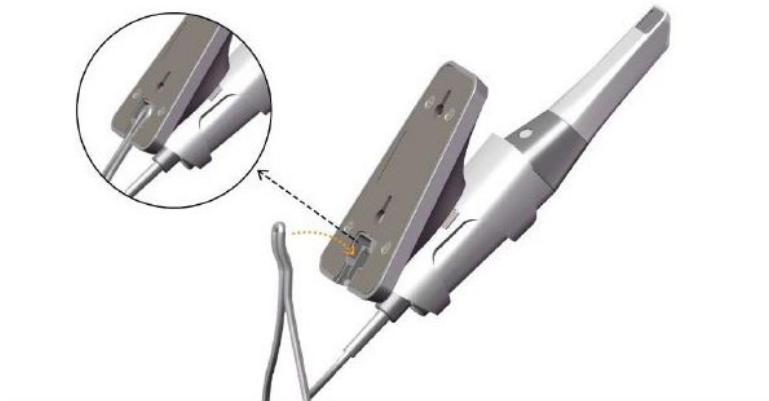- Choose an appropriate length and fix the USB cable to the end of the base of the desktop holder.

- Place the scanner on the desktop holder as shown.


Installing the wall mount holder

To use the holder as a wall mount holder, follow these steps:

Method 1

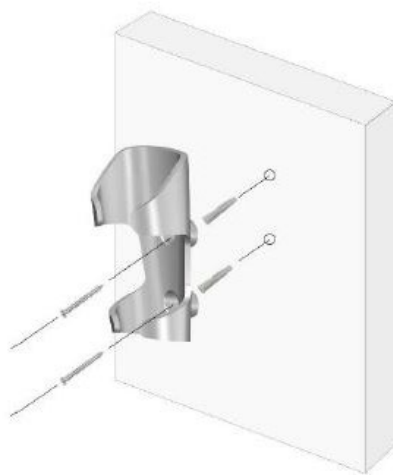- Select an area that you can access easily.
- Insert screws through the holes in the holder bracket to affix it to a solid surface.

Method 2

- Place the base of the holder on the desktop, and fix the holder bracket on the base with screws.

- Select an area that you can access easily.
- Fix the screws to a solid surface.
- Hang the holder to the screws through the holes on the back of the holder base.


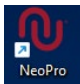
If the holder is not properly installed, there is a risk that the holder can fall off the wall, resulting in damage to the scanner.

5. Getting started


Accessing the NeoPro user interface

To access the NeoPro user interface, follow these steps:

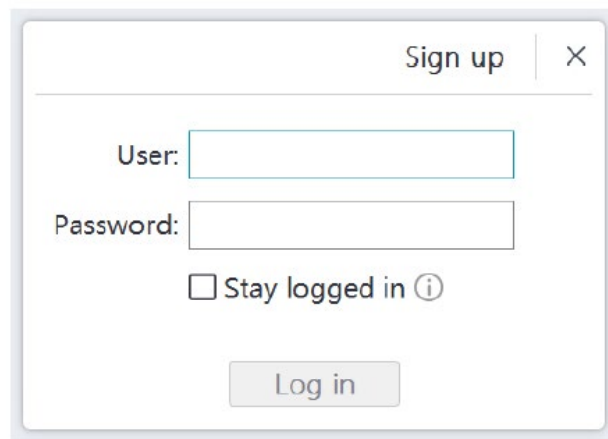1. Double-click the  icon from your desktop.
2. In the NeoPro window, click Login in the title bar to display the login window.

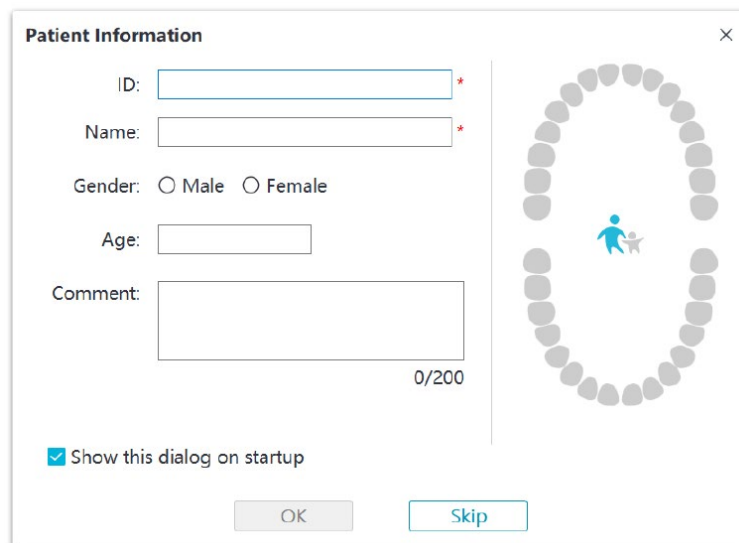3. Click the "Sign up" button to register the organization, and complete the email verification.
4. Type your Neoss NeoConnect account information in the username and password fields, Click the Log in button.
5. The Patient Information dialog box will pop up.
6. Do one of the following:
 - Enter patient information and tooth number information, and click OK.
 - Click Skip and continue without patient information.
7. If the scanner is not activated, the device activation dialog will be displayed. Follow the instructions on the screen to complete device activation.

8. Click the **Option menu**  button and select Preferences
9. Customize the configuration options.

You can now start acquiring 3D models.

Audio prompts

- Scanning sound
When the **Enable scanning sound** option is enabled, your computer will play a continuous sound when you are successfully scanning. If the sound stops, it means the scan has stopped. If you need to continue, please return to the previous scanning area until the scanner resumes scanning and your computer plays a continuous sound. When a bite registration relationship is successfully scanned, your computer will also play a short sound.
- Warning sound
If the cumulative scanning time of the current case exceeds the recommended threshold and your computer may not be able to maintain the peak scanning performance, your computer will play a short warning sound (if your computer does not have speakers, this option will not take effect).

To enable the audio prompts, your computer must have speakers.

Preparing the teeth

- If there is a preparation area, retract the gingiva by gingival restriction cords. And extract the cords just before scanning the preparation.
- Before starting the scan, dry the teeth thoroughly.
- During the scan, re-dry the teeth moderately.

Preparing the scanner

The reusable tip attached to the scanner provides sanitary shield for patients. Always disinfect the body of the scanner and perform high-level disinfection or sterilization on the reusable tip after each use.

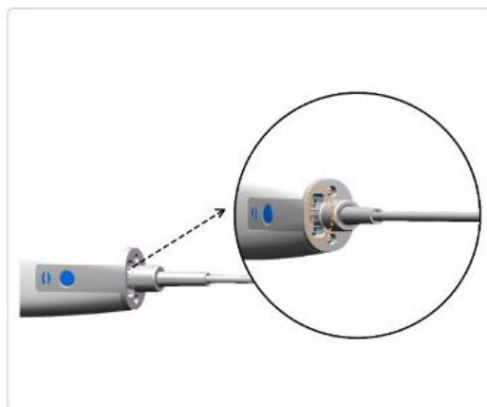
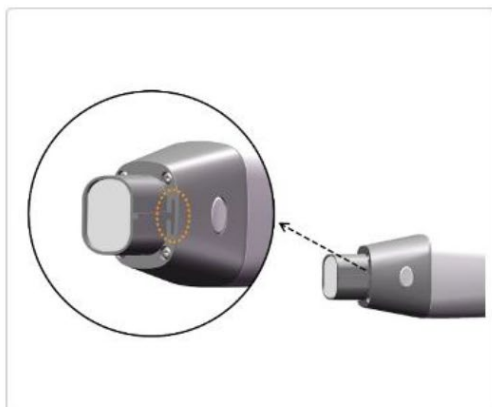
Scanner tips received from the manufacturer are **NOT** sterilized. You must sterilize the tips before the first use.

For detailed information on cleaning, disinfection and sterilization, please refer to the **NeoScan 1000 User Manual: Safety, Regulatory and Technical Specifications**.

Avoid any liquid from leaking into the air outlet near the tip mount or the air inlet at the rear of the scanner (see the figure below), otherwise the scanner may be damaged.

To prepare the scanner, follow these steps:

- Make sure the lens window at the base of the scanner is clean by wiping it with a moist, lint-free cloth or lens tissue.
- Slide the tip onto the scanner as shown below.

Starting scanning

To start scanning, place the tip of the scanner on the surface of the tooth to stabilize the scanner and press the Start Scan button. Wait until a 3D image appears on the 3D model display screen, and then slowly move it along the arch at 0–5mm from the teeth.

During the scanning process, adjust the surgical light to keep the light away from the patient's mouth to avoid interference with the scanner.

Scanning approach

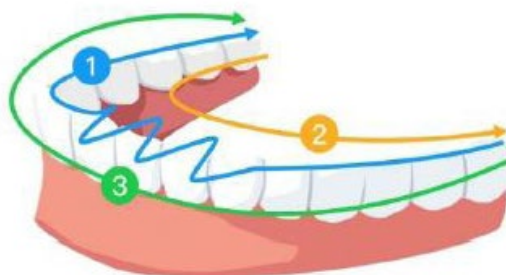
The recommended scanning method is to start with a molar, since it has greater details for easier identification. Change the scanning angle to less than 60 degrees during scanning to allow the surfaces to overlap, if the overlap is too small, the alignment may be lost.

Scanning protocol

The recommended scanning protocol consisting of 3 sweeps: occlusal, lingual and buccal to ensure good data coverage of all surfaces.

It is recommended to start the first sweep from the occlusal surface, you should start with the first molar. The second sweep can scan both the lingual and buccal sides, and the third scan covers the opposite side of the second sweep.

6. Acquiring a 3D model using the common scan workflow

The common scan workflow (default workflow) allows you to scan either a partial or full arch. For either type of scan, you should scan upper jaw, lower jaw, and buccal bite registration. Once this basic information has been scanned, other scan types (such as scan body scanning) can be used to acquire other information.

For example, use a general scanning workflow to scan a 3D model for orthodontics. Scan the complete dental arch to create a 3D model. The user should scan the upper jaw, lower jaw, and buccal bite registration, as well as images of a few millimeters of gum tissue in the lingual and buccal scans. The software will combine these images to create a 3D model, which can then be uploaded to NeoConnect or saved locally.

In some cases, you can acquire 3D models of a single arch (partial or full) and not obtain a buccal bite registration (for example, if there are no teeth in the opposing arch), but it is recommended that you acquire both arches and a buccal bite registration when possible.

To acquire a 3D model, follow these steps:

- Scanning the upper and lower jaw.
- Scan the buccal bite registration.
- Refine and check the 3D model.
- Complete and save the 3D model.

Scanning the upper and lower jaw

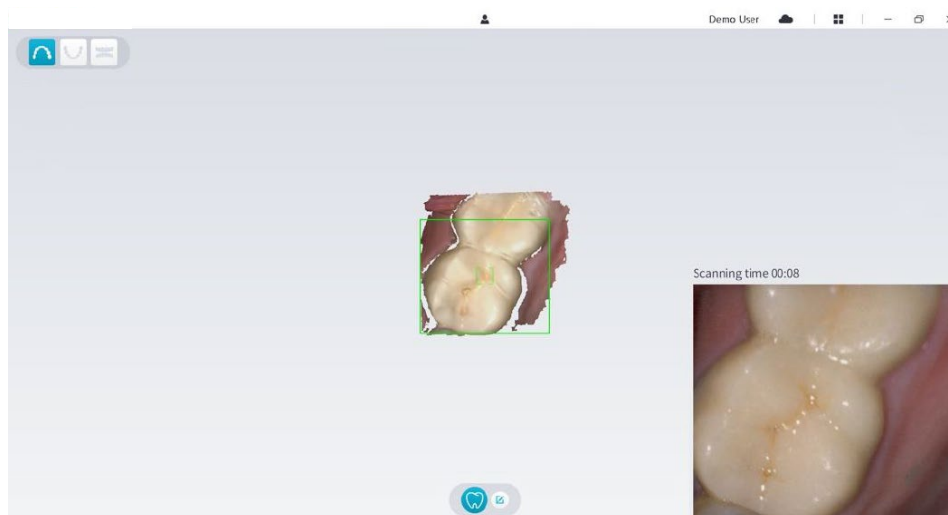
To scan a 3D model of the upper and lower jaw, follow these steps:

- Dry the teeth thoroughly before starting an acquisition.
- On the NeoPro interface, select the Upper Jaw  acquisition mode

OR

Press the mode button on the scanner to select the upper jaw scan mode  .

- Place the tip of the scanner on the surface of the tooth to stabilize the scanner and press the Start Scan button. Wait until a 3D image appears on the 3D model display screen, and then slowly move it along the arch at 0-5mm from the teeth. The image will be automatically scanned and displayed on the 3D model display area.

- Slowly move the tip of the scanner along the occlusal surface to scan the remaining teeth in the arch.

The holes on the 3D model will be displayed in the color specified by the user. It is recommended to scan these areas until the holes disappear.

- When the occlusal surface scan is complete, scan the lingual or buccal surface of the teeth in the arch.
- When the lingual or buccal scan is completed, scan the opposite side of the arch.

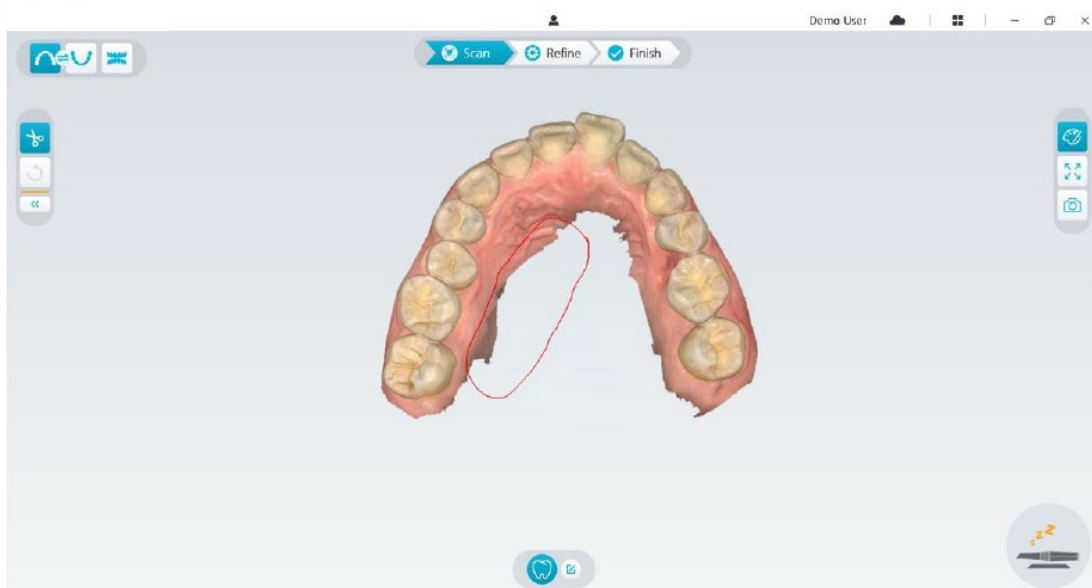
Re-dry the teeth as appropriate throughout the acquisition process.

If you need to remove soft-tissue artifacts, mismatches, or unwanted views during the acquisition, click the **Cut**  tool, then draw a curve that covers the region to be removed on the 3D model. If necessary, rescan the area to fill the holes.

You can click and hold the middle mouse button to rotate the 3D model when using tools such as **Cut** , **Lock**  and **Mark tooth**.

- After scanning the upper jaw, you can continue scanning the lower jaw. On the NeoPro interface, select the Lower Jaw  acquisition mode

OR

Press the mode button on the scanner to select the lower jaw scan mode .

- Repeat steps 3 to 6 until the lower jaw scan is completed.
- Check whether there are obvious holes on the 3D model and rescan if necessary.
- When you confirm that the 3D model of the upper and lower jaw is complete, continue with the buccal bite registration acquisition.

Scanning the buccal bite registration

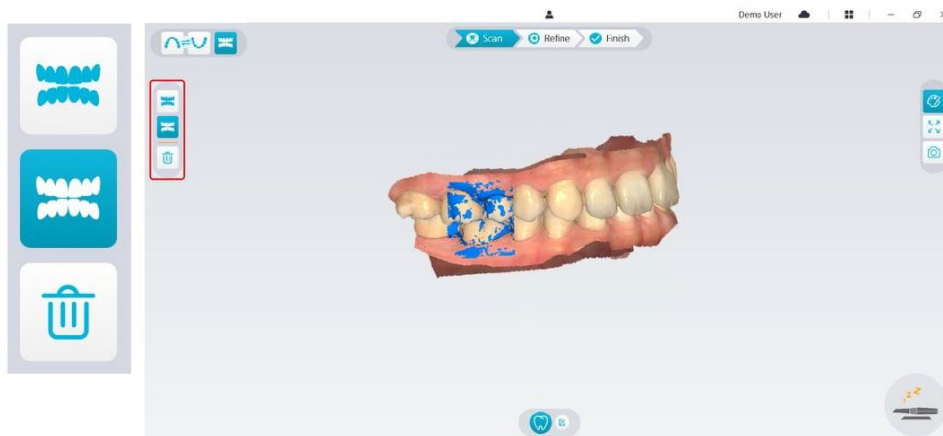
To acquire a buccal bite registration, follow these steps:

- On the NeoPro interface, select the **Buccal Bite Registration**  acquisition mode

OR

Press the mode button on the scanner to select the buccal bite registration mode .

- Place the tip of the scanner into the buccal side in the patient's mouth, then rotate the tip to align with the teeth, close the patient's mouth and confirm that the bite position is correct.
- Press the Start Scan button, slowly move the scanner tip in mesial direction with equal coverage of upper and lower teeth.
- The example below shows a buccal bite registration. You can use the toolbar on the left side to switch between the scanned occlusion views, or delete the scanned occlusion views for rescanning.

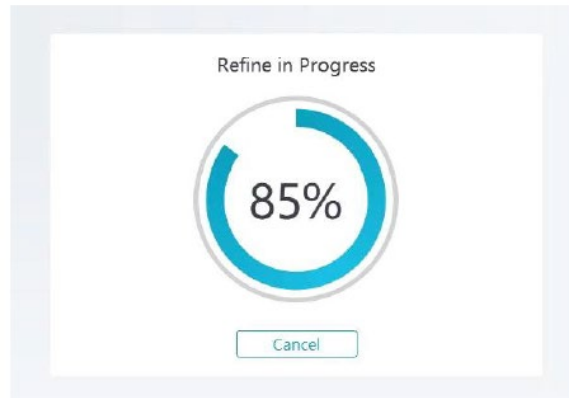
You can scan one or two buccal bite registrations. It is recommended to scan one on the left side and one on the right side of the patient's mouth.

- After scanning the buccal bite registrations, rotate the model and zoom the view to ensure that the bite is accurate and that there are no areas where the bite is mismatched. If necessary, you can delete the scanned occlusion and rescan.

Refining and checking the 3D model

Refining the 3D model allows you to obtain higher accuracy data for further processing. To refine the 3D model, follow these steps:

- Click the **Refine**  button, the refining progress bar will be displayed. Depending on your computer configuration, the refining process may take several minutes.

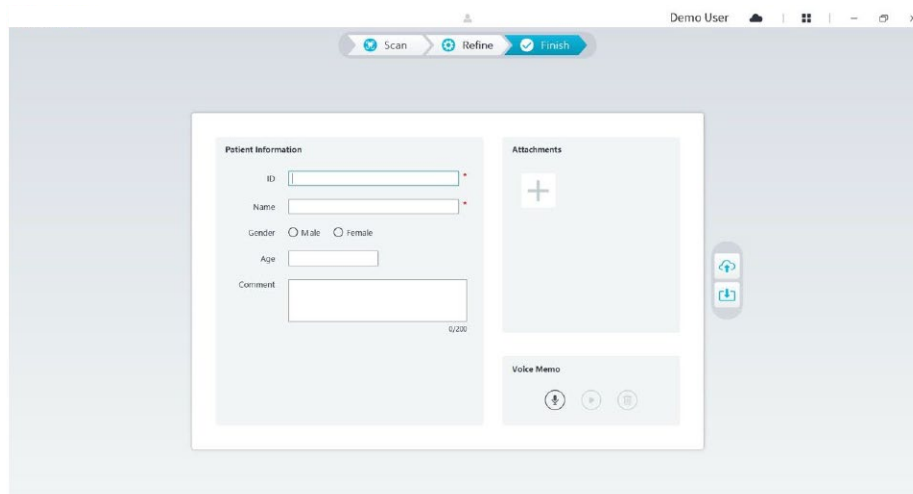
- After the refining is complete, manipulate the refined 3D model using the following methods:
- Click and hold on the 3D model to rotate it.
- Right-click and hold on the 3D model to move it in the window.
- Use the wheel of the mouse to zoom in or zoom out on the 3D model.
- Click the **True color**  button to view the color or monochrome 3D model.
- Click the **Zoom fit**  button to scale the model to its best view.
- Click the **View orientation**  button to see six views of the 3D model.
- Click the **Snapshot**  button to take a snapshot of the 3D model.
- Click **Upper Jaw**  button or **Lower Jaw**  button to hide or display the jaws.
- Click the **Transparency**  button to display the transparency slider, click and drag the slider to adjust the transparency of the 3D model.
- Click **Cut**  button to select and delete unwanted data.
- Click **Intraoral Camera**  button to pick up intraoral images from the scanned data.
- Click the **Quadrant Snapshot**  button to open the Quadrant Snapshot window, which displays multiple views of the model.
- Click the **Occlusion analysis**  button to analyze the occlusion space.
- If you find obvious holes when checking the 3D model, click the **Scan**  button and rescan the areas to fill the holes.
- Repeat steps 1 through 3 until you are satisfied with the 3D model.

Completing and saving the 3D model

To complete the scan and save the 3D model, follow these steps:

- Click the **Finish**  button and the following page will be displayed. If you entered patient information in the previous step, it will be displayed on this page.

- Complete the patient information, if necessary, you can add some attachments to the case.
- Click the **Upload to cloud**  button to upload the case and 3D model data to NeoConnect.
- Click the **Save to file system**  button to save the case and 3D model data to the specified local directory.

7. Acquiring a 3D model using the scan body workflow

You can use the scanner to scan all or part of the dental arch containing the abutment or implant and create a 3D model. When using an existing abutment to scan all or part of the dental arch, use the "common scan" workflow. When you scan all or part of the dental arch that contains the implant, the scan body workflow is used. When using the scan body workflow, the software will copy the 3D model and automatically cut out the implant areas, rescan the tooth jaw with the scan body and create two 3D models: one contains the scan body and the other does not contain the scan body. The user can send these models to the dental laboratory.

In some cases, you can acquire 3D models of a single arch (partial or full) and not obtain a buccal bite registration (for example, if there are no teeth in the opposing arch), but it is recommended that you acquire both arches and a buccal bite registration when possible.

To acquire a 3D model for an implant, follow these steps:

- Scan the upper and lower jaw.
- Scan the buccal bite registration.
- Mark the implant areas
- Install and scan the scan body
- Refine and check the 3D model.
- Complete and save the 3D model.

Scanning the upper and lower jaw

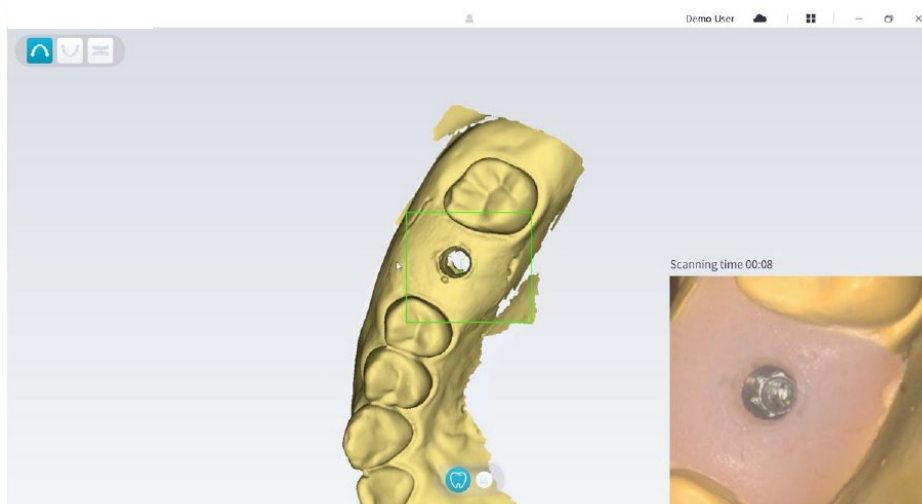
To scan a 3D model of the upper and lower jaw, follow these steps:

- Dry the teeth thoroughly before starting an acquisition.
- On the NeoPro interface, select the **Upper Jaw**  acquisition mode

OR

Press the mode button on the scanner to select the upper jaw scan mode .

- Place the tip of the scanner on the surface of the tooth to stabilize the scanner and press the Start Scan button. Wait until a 3D image appears on the 3D model display screen, and then slowly move it along the arch at 0-5mm from the teeth. The image will be automatically scanned and displayed on the 3D model display area.

- Slowly move the tip of the scanner along the occlusal surface to scan the remaining teeth in the arch.

The holes on the 3D model will be displayed in the color specified by the user. It is recommended to scan these areas until the holes disappear.

- When the occlusal surface scan is complete, scan the lingual or buccal surface of the teeth in the arch.
- When the lingual or buccal scan is completed, scan the opposite side of the arch.

Re-dry the teeth as appropriate throughout the acquisition process.

If you need to remove soft-tissue artifacts, mismatches, or unwanted views during the acquisition, click the **Cut**  tool, then draw a curve that covers the region to be removed on the 3D model. If necessary, rescan the area to fill the holes.

- After scanning the upper jaw, you can continue scanning the lower jaw. On the NeoPro interface, select the **Lower Jaw**  acquisition mode

OR

Press the mode button on the scanner to select the lower jaw scan mode .

- Repeat steps 3 to 6 until the lower jaw scan is completed.
- Check whether there are obvious holes on the 3D model and rescan if necessary.
- When you confirm that the 3D model of the upper and lower jaw is complete, continue with the buccal bite registration acquisition.

Scanning the buccal bite registration

To acquire a buccal bite registration, follow these steps:

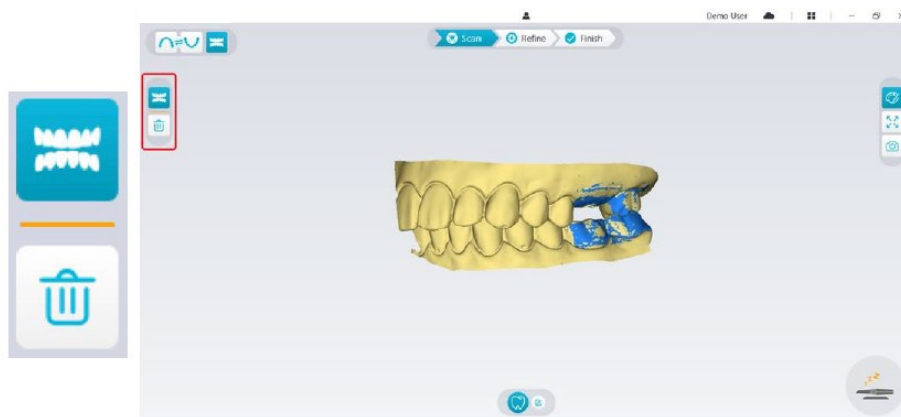
- On the NeoPro interface, select the **Buccal Bite Registration**  acquisition mode

OR

Press the mode button on the scanner to select the buccal bite registration mode .

- Place the tip of the scanner into the buccal side in the patient's mouth, then rotate the tip to align with the teeth, close the patient's mouth and confirm that the bite position is correct.
- Press the Start Scan button, slowly move the scanner tip in mesial direction with equal coverage of upper and lower teeth.

The example below shows a buccal bite registration. You can use the toolbar on the left side to switch between the scanned occlusion views, or delete the scanned occlusion views for rescanning.

You can scan one or two buccal bite registrations. It is recommended to scan one on the left side and one on the right side of the patient's mouth.

- After scanning the buccal bite registrations, rotate the model and zoom the view to ensure that the bite is accurate and that there are no areas where the bite is mismatched. If necessary, you can delete the scanned occlusion and rescan.

Marking the implant areas

After the scan is complete, you can mark the implant areas so that you can rescan the area after installing the scan body. To mark one or more implant areas, follow these steps:

- On the NeoPro interface, select the jaw ( or ) with implant to activate the 3D model.
- Click the **Mark tooth**  button, and then click in the center of the implant. A 3D ball will display on the occlusal surface indicating an implant.

- Rotate the 3D model if necessary, and mark all the implant areas.

You can click and hold the middle mouse button to rotate the 3D model when using tools such as **Cut** , **Lock**  and **Mark tooth**.

- Click the **Return**  button to quit Mark tooth tool.

Installing and scanning the scan body

After installing the scan body, follow the steps below to scan the scan body:

- After making the implant areas, click the **Config Scan** button at the bottom of the NeoPro interface.

- Click the **Add scan body** button in the pop-up window.

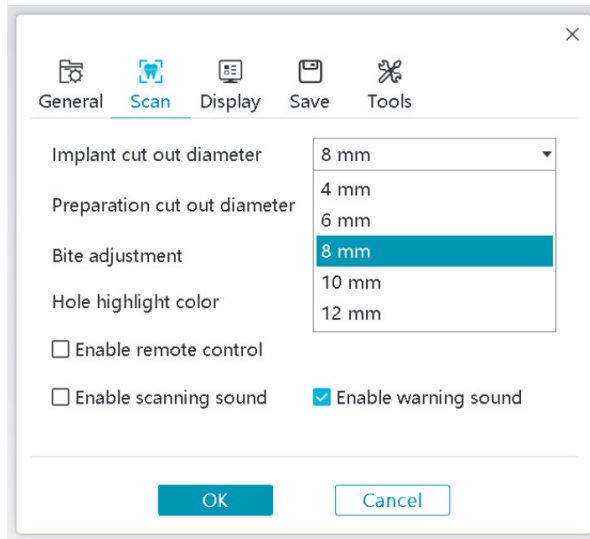
- Click the **Scan Body** button below.


- Check whether the part that the software automatically cut out from the implant areas is sufficient to contain the diameter of the scan body. If the cut out part is not enough to contain the scan body, you can click the **Cut**  button, and cut off the necessary part to cover the scan body. In order to avoid the situation that the

cut out part is not sufficient to contain the scan body, you can adjust the "Implant cut out diameter" option to fit the scan body you choose. After the adjustment, the change will take effect the next time you open the software.

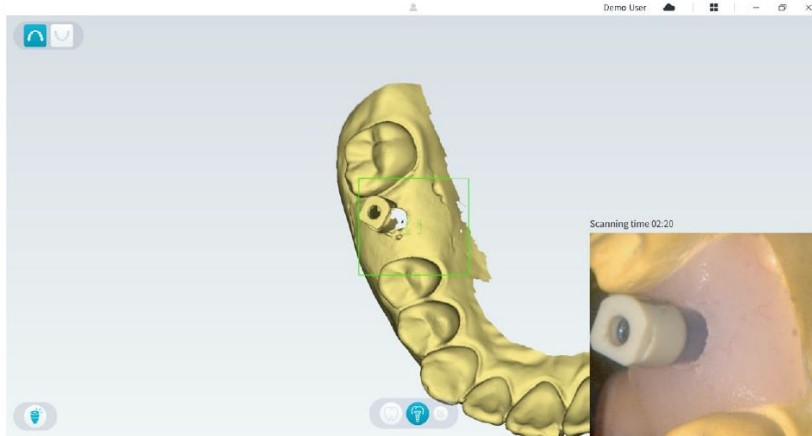- Start the scanner and scan the part of the scan body areas that has been cut out.

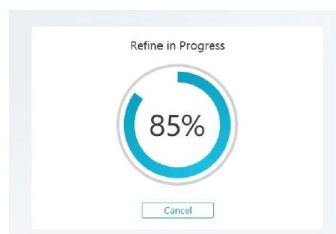
Start the scan body scanning from the surface 1-2 teeth away from the scan body, so that the system can recognize the 3D structure.

- When you have completed the scan body scanning, proceed to the **Refining and Checking the 3D Model step**.

Refining and checking the 3D model

Refining the 3D model allows you to obtain higher accuracy data for further processing. To refine the 3D model, follow these steps:

- Click the **Refine**  button, the refining progress bar will be displayed. Depending on your computer configuration, the refining process may take several minutes.

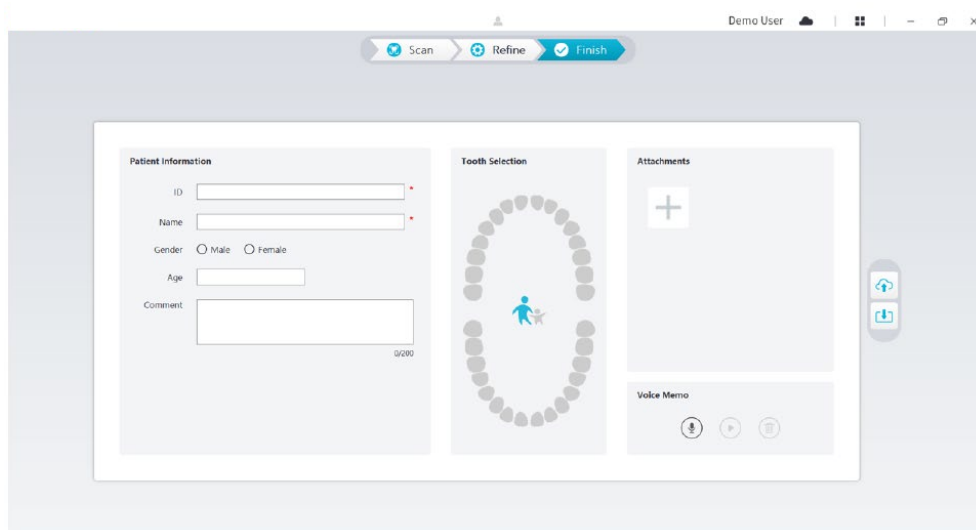
- After the refining is complete, manipulate the refined 3D model using the following methods:
- Click the **Common scan**  button to view the 3D model without scan body.
- Click the **Scan body**  button to view the 3D model with scan body.
- Click and hold on the 3D model to rotate it.
- Right-click and hold on the 3D model to move it in the window.
- Use the wheel of the mouse to zoom in or zoom out on the 3D model.
- Click the **True color**  button to view the color or monochrome 3D model.
- Click the **Zoom fit**  button to scale the model to its best view.
- Click the **View orientation**  button to see six views of the 3D model.
- Click the **Snapshot**  button to take a snapshot of the 3D model.
- Click **Upper Jaw**  button or **Lower Jaw**  button to hide or display the jaws.
- Click the **Transparency**  button to display the transparency slider, click and drag the slider to adjust the transparency of the 3D model.
- Click **Cut**  button to select and delete unwanted data.
- Click **Intraoral Camera**  button to pick up intraoral images from the scanned data.
- Click the **Quadrant Snapshot**  button to open the Quadrant Snapshot window, which displays multiple views of the model.
- Click the **Occlusion analysis**  button to analyze the occlusion space.
- If you find obvious holes when checking the 3D model, click the **Scan**  button and rescan the areas to fill the holes.
- Repeat steps 1 through 3 until you are satisfied with the 3D model.

Completing and saving the 3D model

To complete the scan and save the 3D model, follow these steps:

- Click the **Finish**  button and the following page will be displayed. If you entered patient information in the previous step, it will be displayed on this page.

- Complete the patient information, if necessary, you can add some attachments to the case.
- Click the **Upload to cloud**  button to upload the case and 3D model data to NeoConnect.
- Click the **Save to file system**  button to save the case and 3D model data to the specified local directory.

8. Acquiring a 3D model using the preparation workflow

Depending on your workflow preferences, you can use the preparation workflow. This workflow provides you with two choices:

- You can scan the tooth before preparing, and then take another scan after the tooth is prepared.
- You can import the previously saved data before preparing, and then start to scan the preparation.

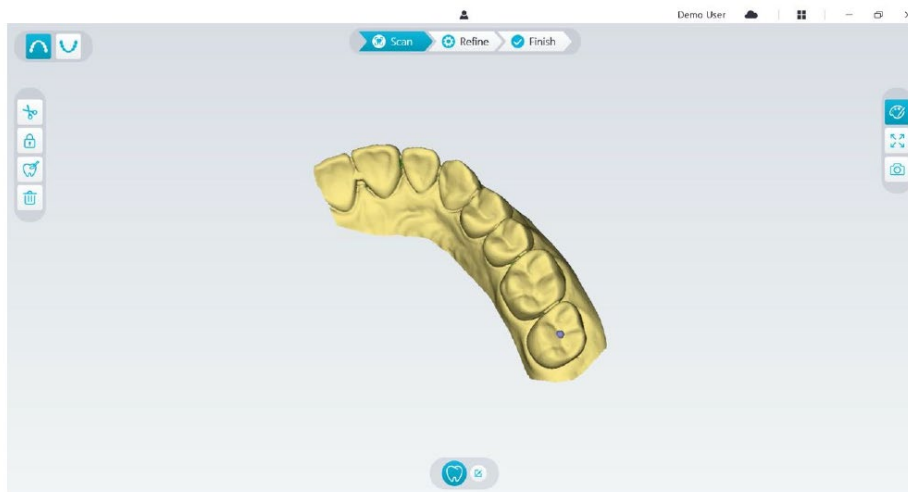
With either option, two 3D models will be created after the scan is completed: one containing the unprepared tooth and the other containing the preparation. You or the lab can use these 3D models for restorations to make them closely resembles the original tooth.

The following steps describe how to combine the previous scan with the new scan of the preparation.

- Import the scanned data before preparing
- Scan the preparation areas
- Refine and check the 3D model.
- Complete and save the 3D model.

Import the scanned data before preparing

1. Click the **Option menu**  button and select Import scan data.
2. Select the .scan file you want to import, and click Open. Wait for the software to import the data, and the 3D model will be displayed after the import is complete.

3. Click the **Mark tooth**  button, and then click in the center of the occlusal surface of the preparation. A 3D ball will display on the occlusal surface indicating a preparation area.
4. Rotate the 3D model if necessary, and mark all the preparation areas.

You can click and hold the middle mouse button to rotate the 3D model when using tools such as Cut , Lock  and Mark tooth .

5. Click the **Return**  button to quit Mark tooth tool.

Scan the preparation areas

After marking the preparation areas, you need to rescan these areas. To scan one or more marked preparation areas, follow these steps:

- Click the **Config Scan** button at the bottom of the NeoPro interface.

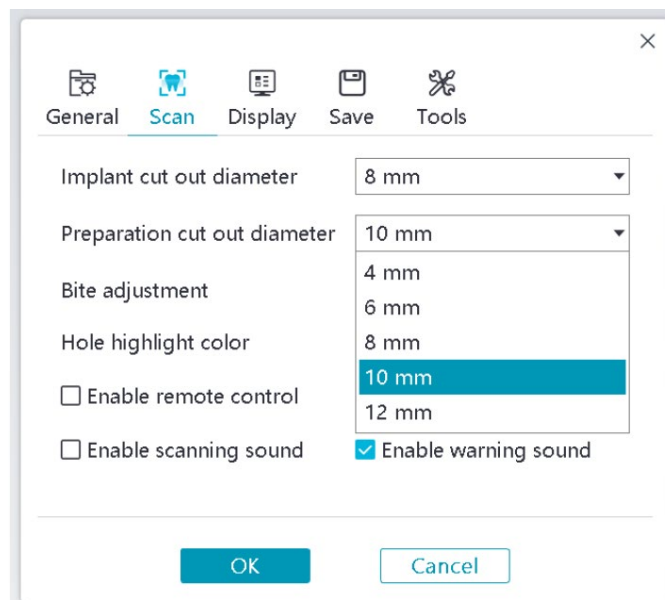
- Click the **Add preparation** button in the pop-up window


- Click the **Preparation** button below.

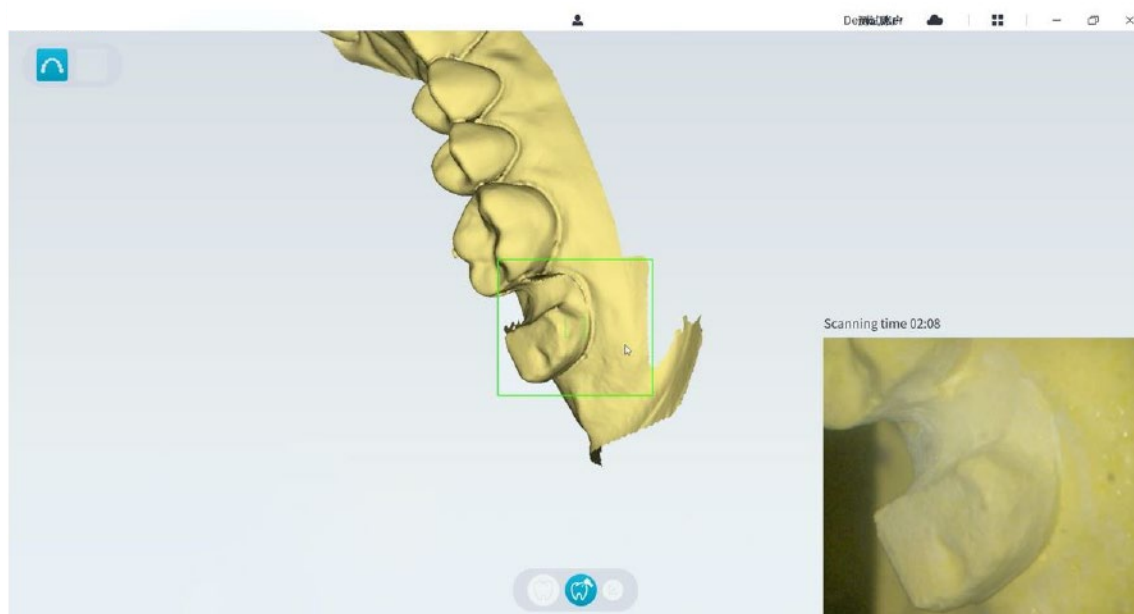- Check whether the part that the software automatically cut out from the preparation areas is sufficient to contain the preparation. If the cut out part is not enough to contain the preparation, you can click the **Cut**  button, and cut off the necessary part to cover the preparation. In order to avoid the situation that the cut out part is not sufficient to contain the preparation, you can adjust the "Preparation cut out diameter" option to fit the preparation. After the adjustment, the change will take effect the next time you open the software.

- Start the scanner and scan the part of the preparation areas that has been cut out.

- If you need to remove soft-tissue artifacts, mismatches, or unwanted views during the acquisition, click the **Cut**  tool, then draw a curve that covers the region to be removed on the 3D model. If necessary, rescan the area to fill the holes.

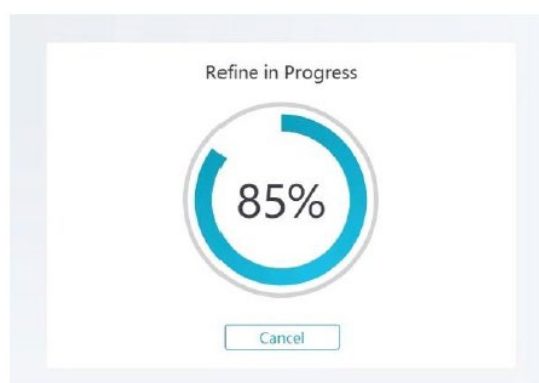
You can click and hold the middle mouse button to rotate the 3D model when using tools such as Cut , Lock  and Mark tooth .

- When you have completed the preparation scanning, proceed to the **Refining and Checking the 3D Model** step.

Refining and checking the 3D model

Refining the 3D model allows you to obtain higher accuracy data for further processing. To refine the 3D model, follow these steps:

- Click the **Refine**  button, the refining progress bar will be displayed. Depending on your computer configuration, the refining process may take several minutes.

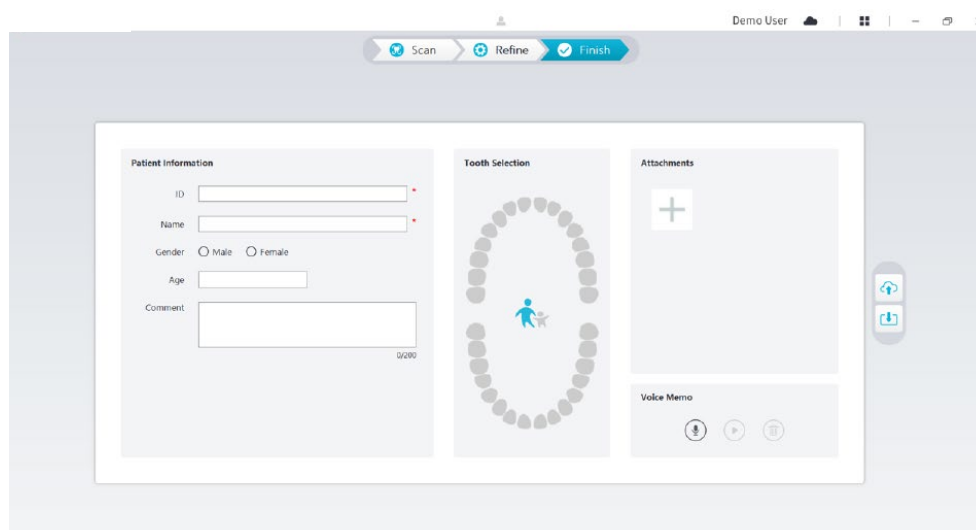
- After the refining is complete, manipulate the refined 3D model using the following methods:

- Click the **Common scan**  button to view the 3D model before preparing.
- Click the **Preparation**  button to view the 3D model after preparing.
- Click and hold on the 3D model to rotate it.
- Right-click and hold on the 3D model to move it in the window.
- Use the wheel of the mouse to zoom in or zoom out on the 3D model.
- Click the **True color**  button to view the color or monochrome 3D model.
- Click the **Zoom fit**  button to scale the model to its best view.
- Click the **Six Views**  button to see six views of the 3D model.
- Click the **Snapshot**  button to take a snapshot of the 3D model.
- Click **Upper Jaw**  button or **Lower Jaw**  button to hide or display the jaws.
- Click the **Transparency**  button to display the transparency slider, click and drag the slider to adjust the transparency of the 3D model.
- Click **Cut**  button to select and delete unwanted data.
- Click **Intraoral Camera**  button to pick up intraoral images from the scanned data.
- Click the **Quadrant Snapshot**  button to open the Quadrant Snapshot window, which displays multiple views of the model.
- Click the **Undercut check**  button to view the undercut areas on the surface of the preparation.
- Click the **Occlusion analysis**  button to analyze the occlusion space.
- If you find obvious holes when checking the 3D model, click the **Scan**  button and rescan the areas to fill the holes.
- Repeat steps 1 through 3 until you are satisfied with the 3D model.

Completing and Saving the 3D Model

To complete the scan and save the 3D model, follow these steps:

- Click the **Finish**  button and the following page will be displayed. If you entered patient information in the previous step, it will be displayed on this page.

- Complete the patient information, if necessary, you can add some attachments to the case.
- Click the **Upload to cloud**  button to upload the case and 3D model data to NeoConnect.
- Click the **Save to file system**  button to save the case and 3D model data to the specified local directory.

9. Maintenance

Cleaning, disinfecting, sterilizing

You must clean and disinfect the scanner, reprocess the tip regularly.

For detailed information on scanner cleaning and disinfecting, and tip reprocessing, please refer to the **NeoScan 1000 User Manual: Safety, Regulatory and Technical Specifications**.

10. Troubleshooting

NeoScan 1000 troubleshooting instructions

Problem Description	Action
There is mismatching and overlap on the 3D model.	Remove mismatched data and excessive tissue using the Cut tool and rescan.
After bite registration, there is a gap or intersection between the upper jaw and the lower jaw.	Delete the incorrect bite view, and rescan. Enable bite adjustment option.
Precision degradation is observed, or images are not well-stitched during acquisition.	Ensure that the lens window at the base of the scanner is clean by wiping it with a moist, lint-free cloth or lens tissue. Use a lens tis-sue or lint-free cloth to remove any dust or water stains. Make sure the tip is firmly installed and there are no dark edges on the live video.
Reconstruction of metallic preparations is sometimes difficult.	Adjust the scanner position (for example: distance or angle) and scan more of the area. Move the surgical light away from the patient to decrease light scatter. Turn on "Shining Surface"button.
The tip is installed but not detected. No live video is displayed, and the Scanner tip not detected icon is displayed at the bottom-right of the interface.	Reinstall the tip, and make sure the tip is in firm contact with the scanner.
Fogging appears on the inner surface of the lens window at the base of the scanner.	Mount a completely dry tip on the scanner, and place the scanner in the holder or set it on the desk, and wait until the fogging fades. If the fogging does not disappear completely after 24 hours, contact your local service provider for assistance. Ensure that the tip is thoroughly dry before mounting on the scanner, and do not use a cloth soaked in disinfectant to clean the scanner.
The scanner does not cast light, and a static preview image is displayed in the video preview screen.	Close the NeoPro software, and then open the NeoPro software again.
After stopping scanning, the scanner is disconnected and cannot be automatically reconnected. Sometimes a dialog pops up saying "Resource temporarily unavailable".	Please try the following steps in order. 1. Check if the scanner is turned off. Press any key on the scanner or take it out of the holder to turn on the scanner again. The scanner startup process may take several seconds. 2. Click the "Scanner is not connected" icon to try to reconnect. 3. Disconnect the USB connector of the scanner from the computer, and then reconnect it to the computer. The scanner will automatically power on and connect.

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
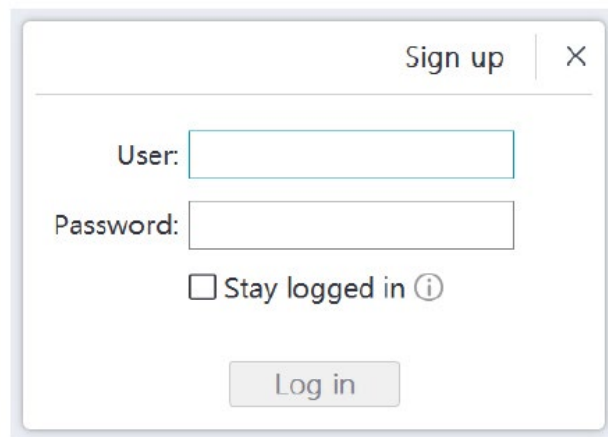
The image shows a 'Sign up' dialog box with a title bar containing 'Sign up' and a close button (X). The dialog contains the following fields and controls:

- User: [text input field]
- Password: [text input field]
- Stay logged in ⓘ
- Log in [button]

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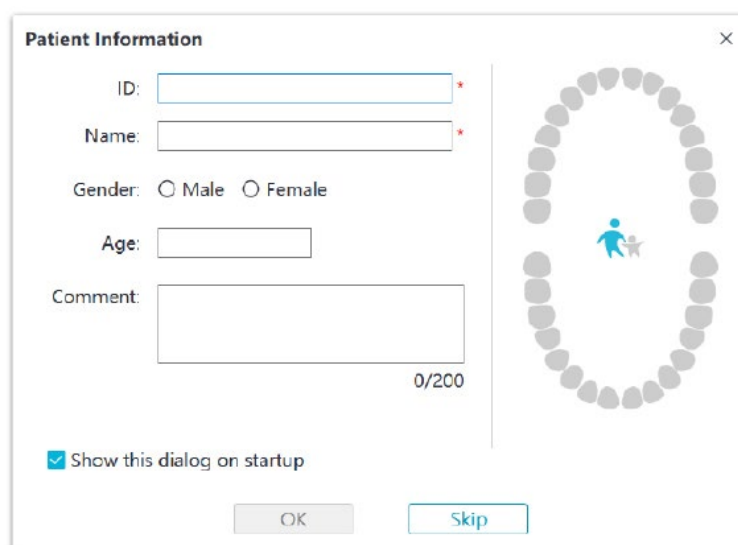The image shows a 'Patient Information' dialog box with a title bar containing 'Patient Information' and a close button (X). The dialog contains the following fields and controls:

- ID: [text input field] *
- Name: [text input field] *
- Gender: Male Female
- Age: [text input field]
- Comment: [text area] 0/200
- Show this dialog on startup
- OK [button]
- Skip [button]

On the right side of the dialog, there is a circular graphic composed of grey dots, with a small blue figure of a person in the center.

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
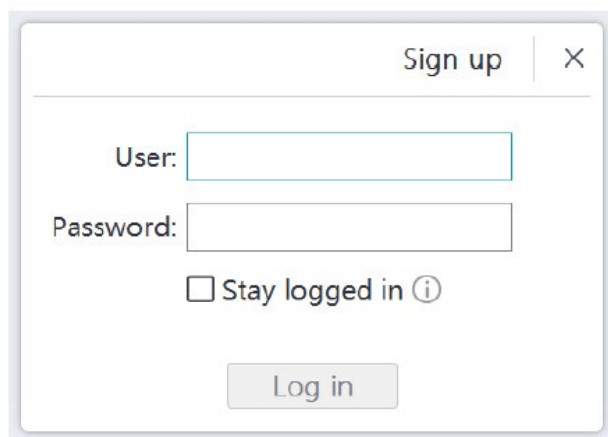
A screenshot of a 'Sign up' dialog box. It has a title bar with 'Sign up' and a close button. The dialog contains two text input fields: 'User:' and 'Password:'. Below the password field is a checkbox labeled 'Stay logged in' with an information icon. At the bottom center is a 'Log in' button.

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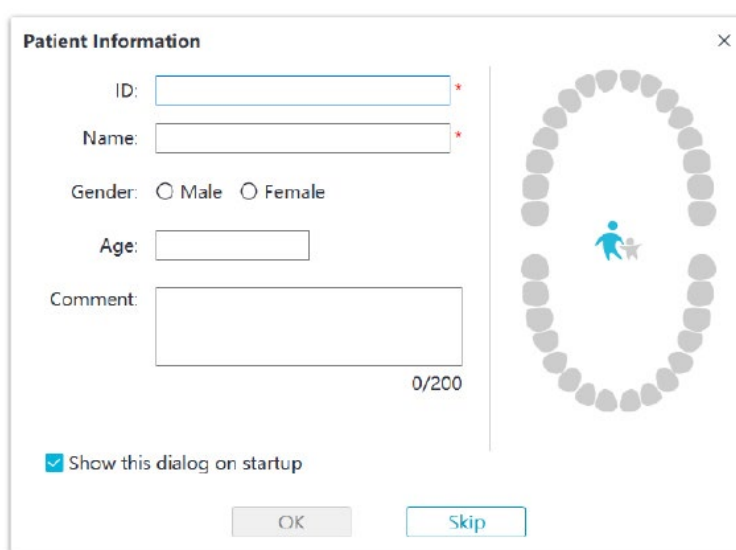A screenshot of a 'Patient Information' dialog box. It has a title bar with 'Patient Information' and a close button. The dialog contains several input fields: 'ID:' (with a red asterisk), 'Name:' (with a red asterisk), 'Gender:' with radio buttons for 'Male' and 'Female', 'Age:', and 'Comment:' (with a character count '0/200'). At the bottom left is a checked checkbox 'Show this dialog on startup'. At the bottom center are 'OK' and 'Skip' buttons. On the right side, there is a circular diagram of teeth with a blue figure of a person sitting in the center.

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
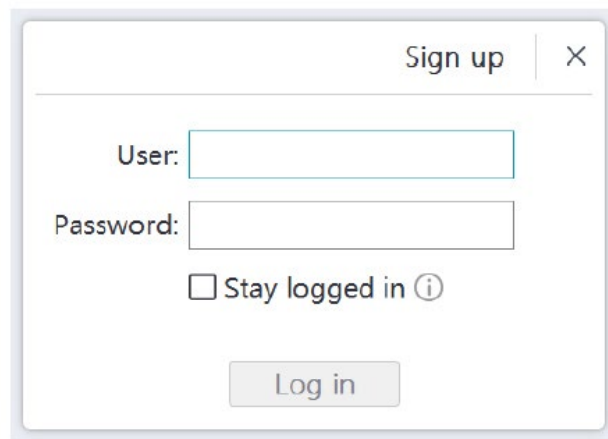
A dialog box titled "Sign up" with a close button (X) in the top right corner. It contains two text input fields: "User:" and "Password:". Below the "Password:" field is a checkbox labeled "Stay logged in" with an information icon (i) to its right. At the bottom center is a "Log in" button.

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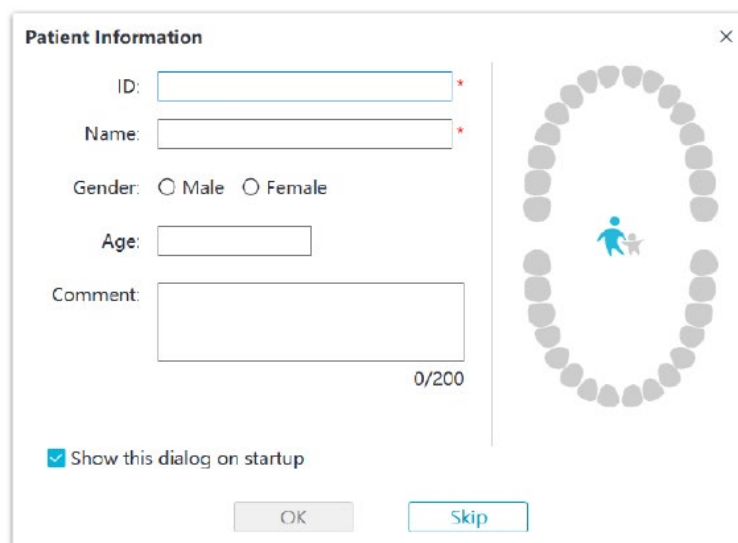A dialog box titled "Patient Information" with a close button (X) in the top right corner. It contains several input fields: "ID:" (with a red asterisk), "Name:" (with a red asterisk), "Gender:" with radio buttons for "Male" and "Female", "Age:" (with a small input field), and "Comment:" (with a larger input field and a "0/200" character count). At the bottom left is a checked checkbox labeled "Show this dialog on startup". At the bottom center are "OK" and "Skip" buttons. On the right side of the dialog is a circular graphic composed of grey dots, with a small blue figure of a person holding a scanner in the center.

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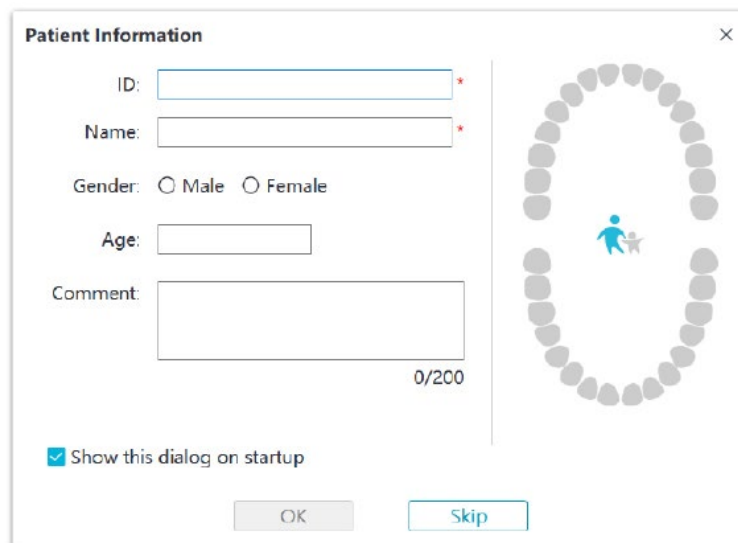
A dialog box titled "Patient Information" with a close button (X) in the top right corner. It contains several input fields: "ID:" (with a red asterisk), "Name:" (with a red asterisk), "Gender:" with radio buttons for "Male" and "Female", "Age:" (with a small input field), and "Comment:" (with a larger input field and "0/200" below it). At the bottom left is a checked checkbox labeled "Show this dialog on startup". At the bottom center are "OK" and "Skip" buttons. On the right side of the dialog is a circular graphic composed of grey dots, with a small blue figure of a person holding a scanner in the center.

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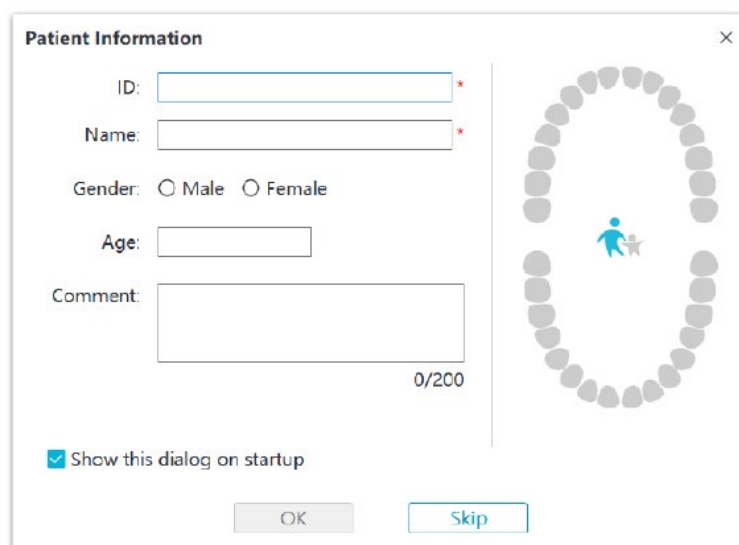
A screenshot of a 'Patient Information' dialog box. It has a title bar with 'Patient Information' and a close button. The dialog contains several input fields: 'ID:', 'Name:', 'Age:', and 'Comment:'. The 'Gender:' field has radio buttons for 'Male' and 'Female'. There are red asterisks next to the 'ID:' and 'Name:' fields. To the right of the text fields is a circular graphic representing a dental arch with a blue figure of a person in the center. At the bottom left, there is a checked checkbox labeled 'Show this dialog on startup'. At the bottom center are 'OK' and 'Skip' buttons.

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
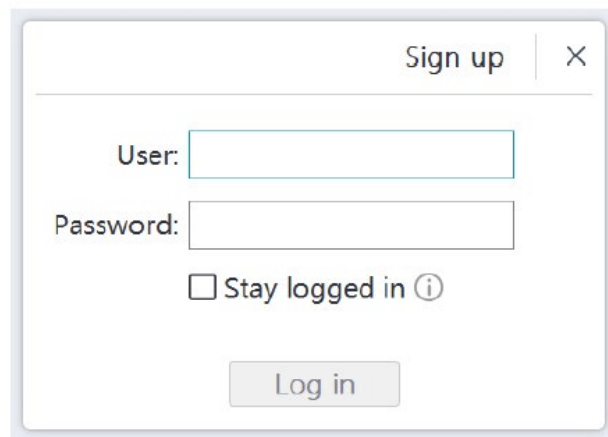
A screenshot of a 'Sign up' dialog box. It has a title bar with 'Sign up' and a close button. The form contains two text input fields labeled 'User:' and 'Password:'. Below the password field is a checkbox labeled 'Stay logged in' with an information icon. At the bottom is a 'Log in' button.

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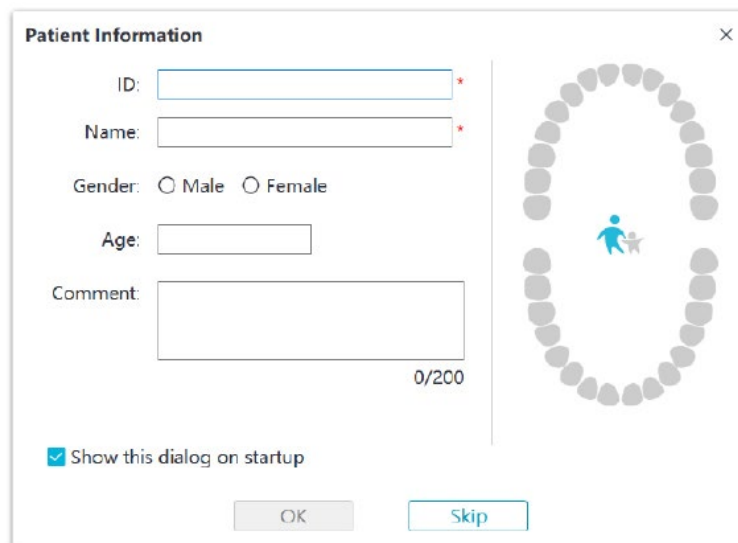A screenshot of a 'Patient Information' dialog box. It has a title bar with 'Patient Information' and a close button. The form contains several input fields: 'ID:' (with a red asterisk), 'Name:' (with a red asterisk), 'Gender:' with radio buttons for 'Male' and 'Female', 'Age:' (with a small input field), and 'Comment:' (with a larger text area and '0/200' character count). At the bottom left is a checked checkbox 'Show this dialog on startup'. At the bottom are 'OK' and 'Skip' buttons. On the right side of the dialog is a circular graphic composed of grey dots, with a small blue figure of a person in the center.

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UNITED KINGDOM & REPUBLIC OF IRELAND

Neoss Ltd
Windsor House
Cornwall Road
Harrogate
HG1 2PW
T: +44 1423 817 733
F: +44 1423 817 744
E: info@neoss.com

UNITED STATES

Neoss Inc.
21860 Burbank Blvd.,
Ste. 190 (North Entrance)
Woodland Hills, CA 91367
T: +1 866 626 3677
F: +1 818 432 2640
E: contact.usa@neoss.com

Our distributors

BAHRAIN

Cigalah Gulf Medical
Building 670, Block 309, Road 919
Salmaniya, Bahrain
T: +973 17232170
E: support@cgmed.com
maha@cgmed.com

CROATIA

Medical Intertrade d.o.o
10431 Sveta Nedelja
HRVATSKA
T: +385 1 33 74 020
F: +385 1 33 36 035
E: dental@medical-intertrade.hr

EGYPT

Dental Future
Building No 5
Mohamed Hassanein Heikal
St. Off Abbas El Akkad Street
1st Floor Flat 3
Cairo, Egypt
T: +20 109 6696449
E: mahmod@dentalfuturekw.com

IRAQ

Eblal Scientific Bureau
Al-Ameerat street
Almansour
Baghdad
Iraq
T: +964 780 411 0110
E: info@eblal.com

JAPAN

Neoss Japan Co.
Y101 Building 5F
No. 1-4 Kanda-Sudacho Chiyodaku
Tokyo 101-0041
T: +81 3 5289 3511
F: +81 3 5289 3512
E: info@neoss.co.jp

JORDAN

Advanced Intelligence for Medical Supplies & Equipment Ltd.
Al-Swefeyyeh Paris str
Alhusainy Complex
Amman 11196
Jordan
T: +962 7 97389535
F: +962 6 5826797
E: f.khaddam@advanceime.com

KINGDOM OF SAUDI ARABIA

Tamer Group
Mohammad Ibn Abdul Wahab Street
Al Ammariya District
P.O. Box 180
Jeddah, 21411
Kingdom of Saudi Arabia
T: +966 12 643 5600
F: +966 12 642 4141
E: alaa.wanly@tamergroup.com

KUWAIT

Dental Future Company
Salmiya Salem el Mubark street
Suhaib complex office 10
Kuwait City
Kuwait
T: +965 51155997
E: info@dentalfuturekw.com

LEBANON

Dentaltech Sarl
Brasilia Street
Hikmeh Building
First Basement
Baabda, Beirut – Lebanon
T: +961 5 950 707
F: +961 5 950 808

NETHERLANDS

Rotor, Medical Supplies B.V.
Rijksweg Zuid 51
NL-6071HV Swalmen
T: +31 46 481 12 02
F: +31 46 481 12 04
E: mail@rotor-medical.com

OMAN

Al Zahrawi Medical Supplies LLC.
Office No: 16 , Al Dar building opposite
Al Zubair automotive, street 61, Ghala
area.
Muscat, Oman
P.O. Box: 1627
T: +968 24587308
F: +968 24587487
E: info.oman@zahrawigroup.com

QATAR

Al Zahrawi Medical LLC
Building No. 36
Zone No. 39
Al Sadd Area
Street No.808
Doha, Qatar, P.O. Box 63065
T: +974 4431 872
F: +974 44365216
E: info.qatar@zahrawigroup.com

SPAIN

Formación, Gestión Y Rendimiento, S.L.
C. Nil Fabra 34-36.
Etslo 2 B
08012. Barcelona
Spain
T: +34 931 408 770
E: info@fgdental.com

SWITZERLAND

Flexident AG
Amlehstr. 22
CH-6010 Kriens
T: +41 41 3104020
E: info@flexident.ch

Imported by:

Neoss AB, Arvid Wallgrens backe 20, SE-413 46 Göteborg
Phone: +46 (0)31 88 12 80 | Email: info@neoss.se

Distributed by:

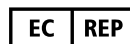
Neoss Ltd. and its subsidiaries as above

Manufacturer:

Alliedstar Medical Equipment Co., Ltd.
No.1-4, Floor 3, Unit 2, Building 3, No.222, West third section,
Waihuan Road, Yanjiang District, Ziyang, 641300, Sichuan, P.R. China
Email: support@allied-star.com

EU Authorized Representatives:

Authorized Representative in the European Community
SUNGO Europe B.V
Olympisch Station 24 , 1076DE Amsterdam, Netherlands

Contact your local representative for product availability details. All products are not available in every market.

15101_0 EN DE IT SV NO ES FR DK NL SV 22-09 © Neoss Limited, 2022. Copyrights, design rights and trademarks, Neoss documents, software and designs may not be reprinted, copied or published in whole or part, without the written authorization of Neoss Limited. Neoss and the N logo are registered trademarks of Neoss Ltd. NeoScan™, NeoPro™ and NeoConnect™ are trademarks of Neoss Ltd.