
















Neoss[®] Custom Prosthetics Torque Guide

	Straight Screw Channel (SSC)		Angulated Screw Channel (ASC)	
Platform	Neo Screw Torque 	Neo Driver 	iGO Screw Torque 	iGO Driver 
SP 3.5 – 6.5	 Neo Abutment Screw, SP 32 Ncm	 Screwdriver Machine	 iGO Abutment Screw, SP 32 Ncm	 iGO Screwdriver Machine
NP 3.25	 Neo Abutment Screw Ti, NP 32 Ncm		 iGO Abutment Screw, NP 20 Ncm	
Access	 Multi-Unit/Access Neo Prosthetic Screw 20 Ncm		 Multi-Unit/Access iGO Prosthetic Screw 20 Ncm	
Multi-Unit				

 **Neo** = Utilizes Neoss standard Screw/Screwdriver

 **iGO** = Utilizes iGO Screw and Screwdriver

16258_0 EN 23-12 © Neoss Group, 2023. Copyrights, design rights and trademarks, Neoss documents, software and designs may not be reprinted, copied or published in whole or part, without the written authorization of Neoss Group. Neoss and the N logo are registered trademarks of Neoss Group. Product images are not necessarily to scale.